






# Skin rejuvenation therapy

RJ-hyaluronic acid + Polylaser derma



<p><b>1. Preparation of the skin</b></p> <p>a) Cleaning (remove grease), maybe as well a soft peeling. No make-up 24 hours prior to the treatment.</p> <p>b) Pre-laser treatment (5 minutes for an area of 15 cm<sup>2</sup>).</p> <p>c) Warm up (e.g. hot towel, vaporisation).</p>	 A photograph showing a patient lying down with their eyes closed, receiving a facial treatment. A person's hands are visible near the patient's face.
<p><b>2. Apply the warm RJ-hyaluronic acid (50-60° C)</b></p> <p>a) Warm up the bottle in hot water.</p> <p>b) Use the syringe to take the required content approx. 0,3 ml for 15 cm<sup>2</sup> from the bottle.</p> <p>c) Paste the warm RJ-hyaluronic acid onto the skin (<u>do not inject!</u>) and perform a massage until the skin absorbed the gel completely.</p>	 A photograph showing a small glass vial with a green cap and a syringe with a needle, used for administering the hyaluronic acid.
<p><b>3. Laser irradiation with the Polylaser derma</b> for approximately 10-15 minutes for an area of 15 cm<sup>2</sup></p>	 A photograph of the Polylaser derma device, a handheld blue and red laser treatment tool, being held by a hand.
<p><b>Important:</b></p> <p>Patient and therapist must wear protection glasses</p>	 <p>Glasses for the therapist</p>  <p>Eyeball glasses for the patient</p>

Repeat the treatment if necessary.